


Tonometer Sleeve Installation Instructions


1. Using the Hex Key provided screw out the screw to ensure enough clearance inside the sleeve for the tonometer to be inserted.

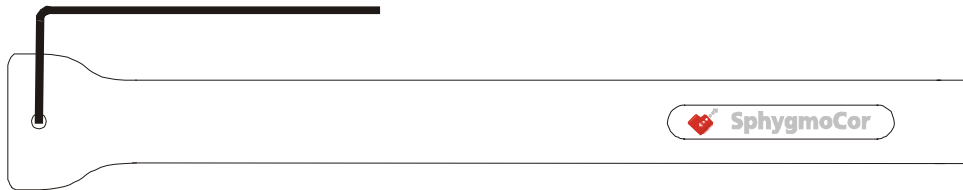

2. Slide the tonometer into the tonometer sleeve.



3. Slide the tonometer through the sleeve allowing the tonometer to protrude from the sleeve 2mm.

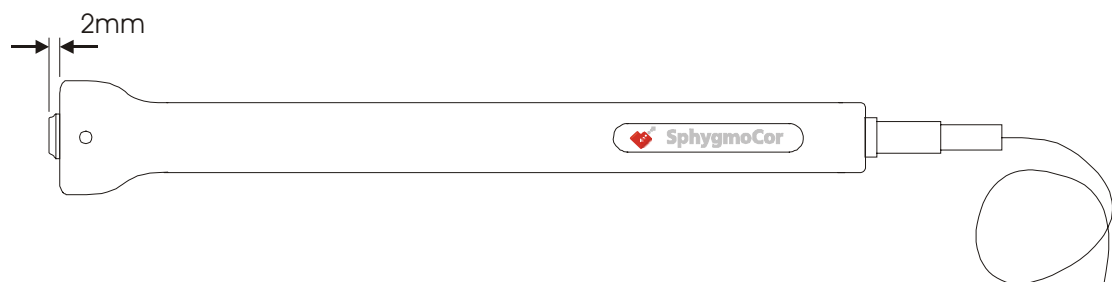

4. Using Hex Key tighten the screw. Once the screw comes in contact with the tonometer only turn the hex key another 1/8 of a turn.

CAUTION: Over tightening of the screw may result in damage to the tonometer.

